

Bassett Healthcare Network

Point of Care

HemoCue Hgb 201+

Quality Control Log

Low Level Control					High Level Control				
Opened Date	Lot #	Opened Exp Date 60 Days	Mfg Exp Date	Acceptable Range (g/dL)	Opened Date	Lot #	Opened Exp Date 60 Days	Mfg Exp Date	Acceptable Range (g/dL)

[illegible]

Refrigerate: Controls expire 60 days after opening. Bring controls to room temperature and mix well before running.

Room Temperature: Controls expire 30 days after opening. Mix well before running.

Cuvettes: Vials expire 3 months after opening. Individually wrapped cuvettes expire on Manufacturer's outdate.

Note: All data is reviewed by Point of Care technical staff on regularly scheduled site visits. The review encompasses information from the last review date to the current review date. Staff performing tests are responsible to ensure that results are within acceptable ranges.

S:\shares3\Lab\POCT\FORMS & LOGS\QC LOGS\HemoCue Hgb QC Log LOW and HIGH.docx