

Nova StatStrip Glucometer: **Cleaning & Disinfection**



REMEMBER THE 2-WIPE RULE!



Clorox Healthcare Bleach
Germicidal Wipes

Info Item #: **52707**

1st wipe to **CLEAN**


2nd wipe to **DISINFECT**

3x vertically and 3x horizontally on all sides
(avoid barcode scanner and electrical connector)

DWELL TIME: 1 MINUTE

(3 MINUTES FOR SUSPECTED OR CONFIRMED C. DIFF AND TB CASES)

Allow to air dry for additional 1 minute before next use.

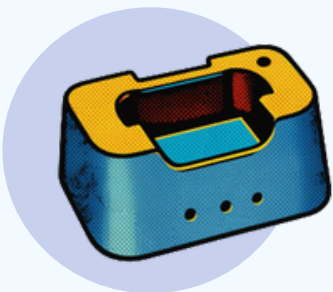

Clean Docking Station AS NEEDED

Use 1 wipe to clean external surface thoroughly.

DWELL TIME: 1 MINUTE

Allow to air dry for additional 1 minute.