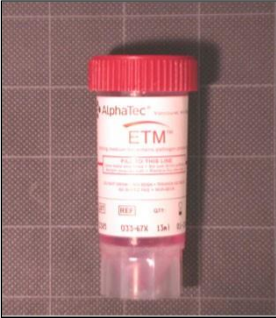
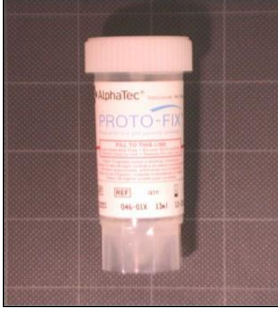


Device	Recommended Uses	Acceptable but Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Copan flocked E-swab 	Aerobic bacterial culture Anaerobic bacterial culture Yeast culture Gram Stain Fungal Stain	Skin Scrapings, send tissue Mycobacteria (AFB) cultures send unpreserved patient sample or tissue Fungal Culture send unpreserved patient sample or tissue Fluid Culture send unpreserved patient sample Tissue culture send unpreserved patient sample or tissue Operative samples send unpreserved patient sample or tissue Rapid Strep testing use EZ II culturette swab Respiratory Culture Wet Preparation	Bacteriology PCR Mycology PCR Viral Culture Stool Culture Chlamydia nucleic acid testing (DNA/RNA) Gonorrhea nucleic acid testing (DNA/RNA) HPV nucleic acid testing Trichomomas nucleic acid testing	Copan flocked E-swab Item Class: Misc. Patient Care Supplies: 554409 Description: SWAB COPAN ESWAB 480C
BD EZ II swab 	Rapid StrepA NAAT COVID, Rapid NAAT Influenza A+B NAAT RSV NAAT	Virus culture, unless immediately placed in M5 buffer Viral PCR unless immediately placed in M5 buffer Mycoplasma PCR unless immediately placed in M5 buffer Ureoplasma PCR unless immediately placed in M5 buffer Aerobic bacterial Culture use Copan flocked E-swab Skin Scrapings, send tissue Mycobacteria (AFB) cultures send unpreserved patient sample or tissue Fungal Culture send unpreserved patient sample or tissue Fluid Culture send unpreserved patient sample Tissue culture send unpreserved patient sample or tissue Operative samples send unpreserved patient sample or tissue Respiratory Syncytial Virus Influenza A+B	Anaerobic culture Bacteriology PCR Mycology PCR Stool Culture Chlamydia nucleic acid testing (DNA/RNA) Gonorrhea nucleic acid testing (DNA/RNA) HPV nucleic acid testing Trichomomas nucleic acid testing	BD EZ II swab Item Class: Misc. Patient Care Supplies: 554409 Description: SWAB BD EZII DUAL 220145
Sterile containers suitable for Aerobic / Anaerobic Culture 	Aerobic Culture Anaerobic culture Gram Stain Fungal Stain Mycobacteria (AFB) stain Mycobacteria (AFB) culture Fungal/Yeast Culture Fluid culture	Virus Culture send unpreserved patient sample	Viral PCR Bacteriology PCR Mycology PCR Wet preparation Cytology Sterile syringe with Needle	A.C.T II (Anaerobic transport) Item Class: Laboratory Supplies: 21608 Description: ACT II Dual Collection System 21608

Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Sterile container suitable for Aerobic Culture 	Aerobic Culture Urine Culture Sputum culture Gram Stain Fungal Stain Mycobacteria (AFB) culture Mycobacteria (AFB) stain Fungal/Yeast Culture Wet preparation Legionella antigen testing (Urine) Streptococcus pneumoniae antigen testing (Urine) Legionella culture Cytology Mycobacterial CX Legionella PCR		Anaerobic Culture Anaerobic Culture, <u>ONLY IF</u> tissue present	Item Class: Laboratory Supplies: 466001 Container Specimen 3 oz 466001 Item Class: Laboratory Supplies: 470106 LAB Centrifuge tube 25/pk 470106
Clean container 	Clostridium difficile Wet Preparation Ova and Parasite Other Fungal Culture	Aerobic culture, send unpreserved patient sample	Aerobic culture Anaerobic Culture Stool culture	Item Class: Laboratory Supplies: 466001 Container Specimen 3 oz 466001 Item Class: Laboratory Supplies: 470138/470139 Tube Lab Specimen Urine - 12 ml Cap Lab Spec tube 500/ca
Cobas PCR Urine Collection Kit Initial sample, <u>not</u> mid-stream 	Chlamydia trachomatis Neisseria gonorrhoeae Trichomonas vaginalis Mycoplasma genitalium Test method: Polymerase Chain Reaction (PCR)		Sexual Abuse Rape Chlamydia Culture / PCR GC Culture / PCR Insufficient Sample Excess Sample HPV testing Aerobic / Anaerobic / Fungal / AFB culture	Cobas PCR Urine Collection Kit Item Class: Laboratory Supplies: TBD Contact BMC laboratory if additional collection kits are needed.

Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Cobas PCR Media Uni Swab Collection Kit Vaginal samples only 	Chlamydia trachomatis Neisseria gonorrhoeae Trichomonas vaginalis Mycoplasma genitalium Test method: Polymerase Chain Reaction (PCR)		Sexual Abuse Rape Chlamydia Culture / PCR GC Culture HPV testing Aerobic / Anaerobic / Fungal / AFB culture	Cobas PCR Media Uni Swab Collection Kit Item Class: Laboratory Supplies: TBD Contact BMC laboratory if additional collection kits are needed.
Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Hologic Aptima Multitest Specimen Collection Kit 	Chlamydia trachomatis Oral or rectal Neisseria gonorrhoeae Oral or Rectal Quest Vaginitis Panel			Hologic Aptima Multitest Collection Kit Item Class: Laboratory Supplies: 60627

Hologic - The "ThinPrep" Pap Test 	Chlamydia trachomatis Neisseria gonorrhoeae HPV with genotyping		Sexual Abuse Rape Chlamydia Culture GC Culture Aerobic / Anaerobic / Fungal / AFB culture	Item Class: Laboratory Supplies: 70136 Thin Prep PAP test / 25 per pack
Alpha Tek ETM 	GI Diarrheal Panel Clostridium difficile Ova & Parasite: • Cyrptosporidium • Giardia • Cyclospora • Entamoeba histolytica		Ova and Parasite - Microscopic	Alpha Tek ETM Item Class: Laboratory Supplies: 900612 Description: PARASET ETM ALPHATEC

Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Alpha Tek PROTO CLR 	Ova and Parasite - Microscopic (Contact Microbiology at 3707 to place the order) Can use Green ECOFix sent to Mayo Medical labs		Clostridium difficile GI Diarrheal Panel Aerobic / Anaerobic / Fungal / AFB culture	Alpha Tek PROTO CLR Item Class: Laboratory Supplies: 400405 Description: PROTOFIX CLR 4602CS
Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
	Aerobic Anaerobic Blood Culture	Fluid Culture	Fungal Culture Mycobacteria culture	Item Class: Laboratory Supplies: 59268 BACTALERT Aerobic SA Blue Item Class: Laboratory Supplies: 59269 BACTALERT Anaerobic Pink

Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
	Fungal Blood Culture Mycobacteria Blood Culture		Aerobic Culture Anaerobic culture	Item Class: Laboratory Supplies: 51582 Vacutainer Green 6ml
Device	Recommended Uses	Not Recommended	NEVER USED	Device name /Item Class & Stock #/Warehouse Description
Copan UTM-RT  	COVID (In-house), Nasal* Respiratory Panel PCR, NP** Herpes & Varicella DNA probe PCR Viral PCR *COVID requires nasal swab **Respiratory Panel requires nasopharyngeal swab			Item Class: Laboratory Supplies