
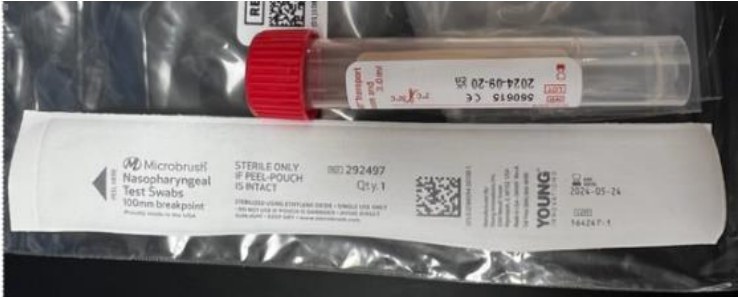


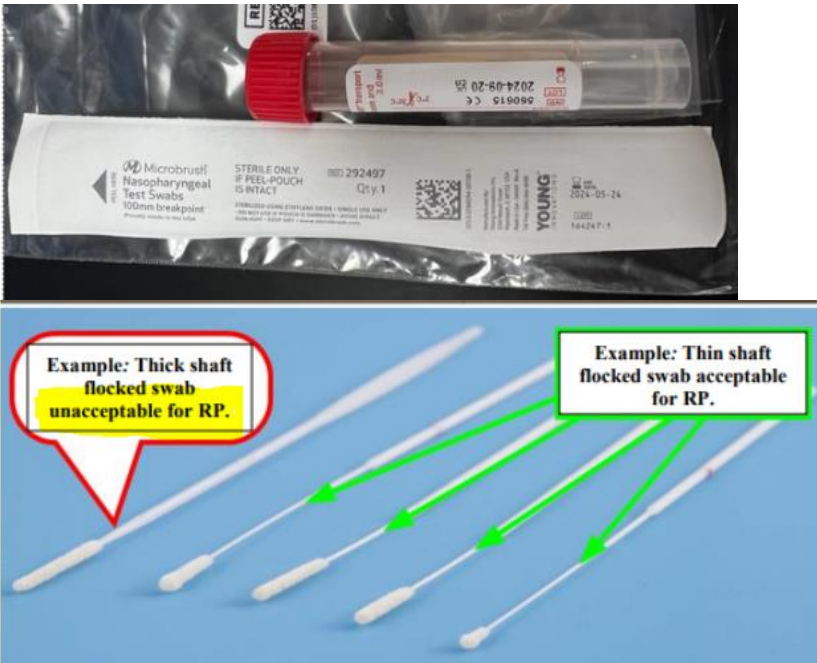

# Influenza, RSV, Respiratory Panel and Covid Samples

Test Name	Picture of Swab	Swab/Container Description	Temperature Stability	Special Instructions
<b>Flu/RSV</b> <b>(Influenza AND Respiratory Syncytial Virus)</b> <b>LAB7110</b>		<b>THIN SHAFT SWAB</b> <b>Viral Transport Media (VTM)- Red top tube with pink liquid inside</b>	<b>Frozen for 14 days</b>	<b>For patients BETWEEN 19-59y/o</b> <b>(Sample sent to Mayo for testing)</b>
<b>RSV</b> <b>(Respiratory Syncytial Virus)</b> <b>LAB2104</b>	 <b>OR</b> 	<b>1 eSwab (Warehouse #51572)</b>  <b>Or</b>  <b>1 EZII Set (Warehouse #51571)</b>	<b>8hr Room Temp/24 Hours Refrigerated</b> <b>Or</b> <b>2hr Room Temp/24hr Refrigerated</b>	<b>Testing done in-house</b> <b>ONLY for patients 0-18yr or 60yr+</b>

# Influenza, RSV, Respiratory Panel and Covid Samples

<p><b>Flu (Influenza A&amp;B)</b> <b>LAB272</b></p>	 <p style="text-align: center;"><b>OR</b></p>	<p><b>1 eSwab (Warehouse #51572)</b></p> <p style="text-align: center;"><b>Or</b></p> <p><b>1 EZII Set (Warehouse #51571)</b></p>	<p><b>72 Hours Refrigerated</b></p>	<p><b>Testing done in-house</b></p>
<p><b>Covid (Pre-Procedure ONLY)</b> <b>LAB2019</b></p>		<p><b>Viral Transport Media (VTM)- Red top tube with pink liquid inside</b></p>	<p><b>96 Hours Refrigerated</b></p>	<p><b>Testing done at MIBH Cooperstown</b></p>

# Influenza, RSV, Respiratory Panel and Covid Samples

<p><b>Respiratory Panel, PCR, Nasopharyngeal</b></p> <hr/> <p><b>LAB1844</b></p>	 <p>The image shows a box of Microbrust Nasopharyngeal Test Swabs. Below the box, two examples of swabs are shown on a blue background. A red callout points to a swab with a thick shaft, labeled 'Example: Thick shaft flocked swab unacceptable for RP.' A green callout points to a swab with a thin shaft, labeled 'Example: Thin shaft flocked swab acceptable for RP.'</p>	<p><b>Nasopharyngeal Swab placed in M4 Media (VTM, UTM, M4-RT are all acceptable. <b>MUST BE THIN-SHAFTED SWAB</b></b></p>	<p><b>Refrigerated 72 hours</b></p>	<p><b>Tested at MAYO</b></p>
<p><b>Covid Rapid</b></p> <hr/> <p><b>LAB2019</b></p>	 <p>The image shows the packaging for a BD BBL EZII Set. The top part shows the sealed package. The bottom part shows the components: a red cap, a swab, and a tube.</p>	<p><b>1 EZII Set (Warehouse #51571)</b></p>	<p><b>Must be tested within <b>45 minutes</b> of collection</b></p>	<p><b>Approved testing sites</b></p>