BMC, CRH, LFH, HRK, OCH, FTT, FOX, FCC

## **Performing Location(s)**

BMC, CRH, LFH, HRK, OCH, FTT, FOX, FCC

# **Specimen Type**

Serum or Plasma

#### **Preferred Container**

Gold or Red top

#### **Alternate Container**

Green top

#### **Minimum Volume to Submit for Testing**

1 mL Serum or Plasma Multiple test can be performed from this volume. For question please contact the lab at 547-3975.

## **Transportation Needs**

Deliver specimen to laboratory within 1 hour of collection if unspun. If not possible see "Storage Requirements".

## **Storage Requirements**

Sample tubes should be centrifuged within 2 hours of collection Gold tops are stable in original tubes for 7 days DO NOT ALIQUOT Aliquot Red tops after centrifuging

See Causes of Rejection for temperature requirements.

#### **Causes for Rejection**

Mislabeled or unlabeled specimen
Less than 50% draw for vacutainer tubes
> 7 days refrigerated
> 7 days room temperature
> 1 year frozen
Hemolysis > 500 mg/dL HgB
Lipemia > 1000 mg/dL Trig
Bilirubin > 40 mg/dL Bili
EDTA, Citrate

#### **Reference Values**

Population	Units	Reference Range	<b>High Critical Value</b>	<b>Low Critical Value</b>
Less than 1 week	mg/dL	7.0-12.0	13.0	< 6.0

General (> 1 week) mg/d	8.6-10.3	> 13.0	< 6.0
-------------------------	----------	--------	-------

# **Available STAT**

Yes

Methodology Beckman AU/Dx Series

# **CPT Code**

82310