

LAB20 Hepatic Function Panel

BMC, CRH, LFH, HRK, OCH, FTT, FOX, FCC

Performing Location(s)

BMC, CRH, LFH, HRK, OCH, FTT, FOX, FCC

Specimen Type

Serum or Plasma

Preferred Container

Gold or Red top tube

Alternate Container

Green top tube

Minimum Volume to Submit for Testing

0.5 mL Serum or Plasma

Multiple test can be performed from this volume.

For question please contact the lab at 547-3975.

Transportation Needs

Deliver specimen to laboratory within 1 hour of collection if unspun.

If not possible see "Storage Requirements".

Storage Requirements

Sample tubes should be centrifuged within 2 hour of collection followed by transfer of the serum (red or gold top) or plasma (green top) to a labeled plastic, aliquot tube.

See Causes of Rejection for temperature requirements for the Individual Components.

Causes for Rejection

Mislabeled or unlabeled specimen

Less than 50% draw for vacutainer tubes

Serum/Plasma not separated from cells within 2 hours of collection

Serum/Plasma - Stability depends on the component

Reference Values

See individual components of the panel

Available STAT

Yes

Methodology

Beckman AU/Dx Series

CPT Code

80076