BMC, HRK, FOX, FCC

## **Performing Location(s)**

BMC, HRK, FOX, FCC

# **Specimen Type**

Serum or Plasma

#### **Preferred Container**

Gold or Red top tube

#### **Alternate Container**

Green top

## **Minimum Volume to Submit for Testing**

1 mL Serum or Plasma Multiple test can be performed from this volume. For question please contact the lab at 547-3975.

#### **Transportation Needs**

Deliver specimen to laboratory within 1 hour of collection if unspun. If not possible see "Storage Requirements".

## **Storage Requirements**

Sample tubes should be centrifuged within 2 hours of collection Gold tops DO NOT ALIQUOT Aliquot Red tops
See Causes of Rejection for temperature requirements.

#### **Causes for Rejection**

Mislabeled or unlabeled specimen
Less than 50% draw for vacutainer tubes
> 5 days refrigerated
> 24 hours room temperature
Heparinized plasma stable 3 days at room temperature
Frozen at -70°C for 3 months
Hemolysis > 50 mg/dL HgB
Lipemia > 1000 mg/dL Trig
Bilirubin > 40 mg/dL Bili
EDTA, NaF, or Oxalate

#### **Reference Values**

Population	Units	Reference Range

< 1 week old	U/L	292-658
1 week - 1 month old	U/L	131-282
1 month - 12 years old	U/L	101-237
General (> 12 years old)	U/L	140-271

# **Available STAT**

Yes

Methodology Beckman AU/Dx Series

# **CPT Code**

83615