

## POC1620 POC PT/INR

---

Bassett Healthcare Network POC Testing

### Performing Location(s)

Bassett Healthcare Network POC Testing. Only available at POC Approved Testing Locations.

### Specimen Type

Capillary

### Preferred Container

Capillary fingerstick

**Allow the puncture site to air dry**

**Do not wipe away the first drop of blood. Use first drop for testing**

Venous sample collection: Collect into a non-anticoagulated syringe. Discard the first four drops of blood collected; immediately apply one drop of blood to test strip

### Volume Required

20 uL

### Reference Values

Test	Value
Prothrombin Time (PT)	10.1 – 13.2
INR	0.9 – 1.2
INR (Routine Oral Anticoagulation)	2.0 – 3.0
INR (Mechanical Valves)	2.5 – 3.5
<b>Critical Value INR: &gt; 4.4</b>	

### Confirmatory Testing

Venous Confirmation: A Laboratory confirmation is required for INR results >6.0

The order for the confirmatory PT/INR is automatic and all INRs >6.0 require a sample submitted to the Clinical Laboratory for confirmatory testing.

The Beaker label will print to the specimen label printer upon verifying the results.

### Available STAT

Only available at POC Approved Testing Locations.

### Methodology

CoaguChek XS Plus

### CPT Code

85610