BMC, FOX, FTT, CRH, LFH, OCH

### **Performing Location(s)**

BMC, FOX, FTT, CRH, LFH, OCH

# **Specimen Type**

Serum

#### **Preferred Container**

Red top tube

## **Minimum Volume to Submit for Testing**

0.5 mL Serum

Multiple test can be performed from this volume.

For question please contact the lab at 547-3975.

#### **Storage Requirements**

Sample tubes should be centrifuged within 1 hour of collection followed by transfer of the serum to a labeled plastic, aliquot tube.

### **Transportation Needs**

Deliver specimen to laboratory within 1 hour of collection if unspun.

If not possible see "Storage Requirements".

#### **Causes for Rejection**

Mislabeled or unlabeled specimen

Less than 50% draw for vacutainer tubes

Anything other than a red top tube

Hemolysis > 800 mg/dL HgB

Lipemia > 750mg/dL Trig

Bilirubin > 30 mg/dL Bili

- > 1 month refrigerated
- > 24 hours room temperature
- > 3 months frozen

Heparin, EDTA, NaF, Citrate

#### **Reference Values**

Population	Units	Therapeutic Range	<b>High Critical Value</b>
General	ug/mL	10.0 - 20.0	25.0

## **Available STAT**

Yes

**Methodology** Beckman AU/Dx Series

**CPT Code** 

80185